



smiths medical

bringing technology to life

Warmer  
patients.  
Better  
outcomes.

SNUGGLE WARM®  
CONVECTIVE WARMING  
BLANKETS REFERENCE  
CARDS

TEMPERATURE  
MANAGEMENT

LEVEL 1®

At Smiths Medical, we believe that innovation goes well beyond products. That is why we collaborate closely with clinicians to ensure that all of our Level 1® products and services satisfy our customers' needs. **As a leader in temperature management, Level 1® convective warming products provide a comprehensive selection of innovative blanket styles for a wide variety of surgical procedures.** Throughout the perioperative period, Snuggle Warm® Convective Warming Blankets are proven and dependable, improving patient outcomes every day.

# Snuggle Warm® Convective Warming Blankets are:

LEVEL 1®

- Made of soft fabric-like material that is comfortable next to patients' skin — no hot plastic
- Strong and durable — resist tears, punctures and fluids
- Designed to conform to the patient, reducing the need for tie-downs
- Made of foldable blanket material to allow for easy access to the patient
- Equipped with precisely positioned air holes and channel design to maximize heat delivery
- Latex-free and radiotranslucent
- Available in multiple configurations for a broad set of clinical applications



**SWU-2001**  
Full Body  
Blanket



**SWU-2002**  
Pediatric Full  
Body Blanket



**SWU-2003**  
Upper Body  
Blanket



**SWU-2004**  
Lower Body  
Blanket



**SWU-2007**  
Tube  
Blanket



**SWU-2008**  
Sterile  
Cardiac  
Blanket



**SWU-2009**  
Pediatric  
Underbody  
Blanket



**SWU-2010**  
Small Upper  
Body Blanket



**SWU-2011**  
Large  
Pediatric  
Underbody  
Blanket



**SWU-2013**  
Adult  
Underbody  
Blanket



**SWU-2013**  
Adult  
Underbody  
Blanket with  
Arm Openings



**SWU-2014L**  
Left Lateral  
Access Blanket



**SWU-2014R**  
Right Lateral  
Access Blanket



**SWU-2016**  
Full Body Split  
Access Blanket



**SWU-2018**  
Multi-Access  
Blanket




**SWU-2019**  
Poncho  
Blanket with  
Arm Openings

## Adult Blankets

|    |                  |   |
|----|------------------|---|
| 1  | <b>SWU-2001</b>  | Full Body Blanket                         |
| 2  | <b>SWU-2003</b>  | Upper Body Blanket                        |
| 3  | <b>SWU-2004</b>  | Lower Body Blanket                        |
| 4  | <b>SWU-2007</b>  | Tube Blanket                              |
| 5  | <b>SWU-2008</b>  | Sterile Cardiac Blanket                   |
| 6  | <b>SWU-2010</b>  | Small Upper Body Blanket                  |
| 7  | <b>SWU-2013</b>  | Adult Underbody Blanket                   |
| 8  | <b>SWU-2113</b>  | Adult Underbody Blanket with Arm Openings |
| 9  | <b>SWU-2014L</b> | Left Lateral Access Blanket               |
| 10 | <b>SWU-2014R</b> | Right Lateral Access Blanket              |
| 11 | <b>SWU-2016</b>  | Full Body Split Access Blanket            |
| 12 | <b>SWU-2018</b>  | Multi-Access Blanket                      |
| 13 | <b>SWU-2119</b>  | Poncho Blanket with Arm Openings          |

## Pediatric Blankets

|    |                 |                                   |
|----|-----------------|-----------------------------------|
| 14 | <b>SWU-2002</b> | Pediatric Full Body Blanket       |
| 15 | <b>SWU-2009</b> | Pediatric Underbody Blanket       |
| 16 | <b>SWU-2011</b> | Large Pediatric Underbody Blanket |

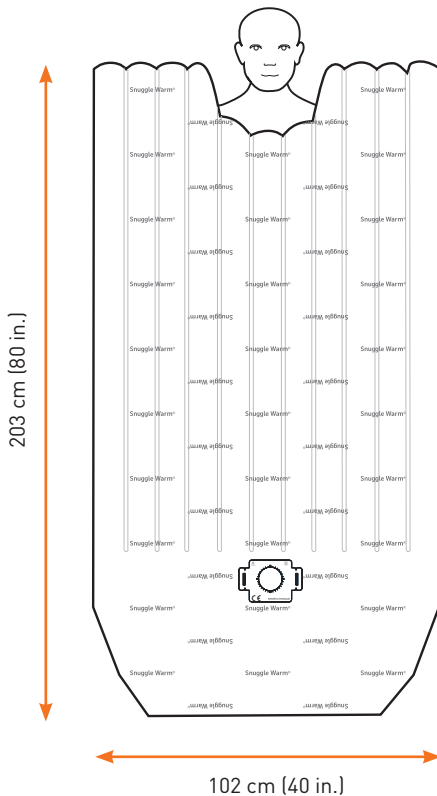

# SWU-2001

## Full Body Blanket

102 x 203 cm

40 x 80 in.


Includes non-inflating foot drape

### **Recommended for use in these procedures:**

- Pre-operative and post-operative warming
- Bariatric
- Neurological
- Ear, nose and throat
- Trauma

### **Key attributes of the SWU-2001:**

- Large size ensures maximum coverage for all patient sizes

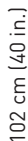

## Upper Body Blanket

203 x 102 cm

80 x 40 in.

Includes non-inflating flaps


### **Recommended for use in these procedures:**

- Lower extremities (hips, legs, knees, ankles)
- Gynecology
- Urology
- Open abdominal
- Laparoscopic
- Bariatric

### **Key attributes of the SWU-2003:**

- Accordion-style chest panel accommodates large and small patients
- Clear plastic drape captures and delivers warm air around the patient's head



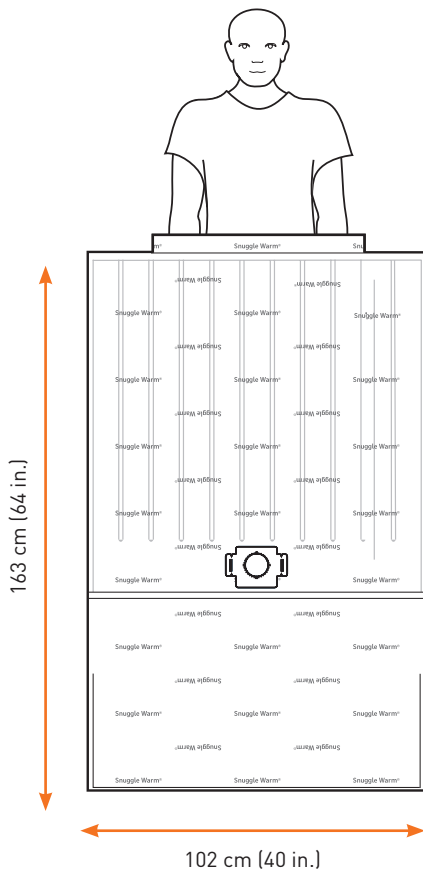

## Lower Body Blanket

102 x 163 cm

40 x 64 in.


Includes non-inflating foot drape

### **Recommended for use in these procedures:**

- Upper extremity
- Shoulder and neck
- Ear, nose and throat

### **Key attributes of the SWU-2004:**

- Non-inflating foot drape protects patient's feet

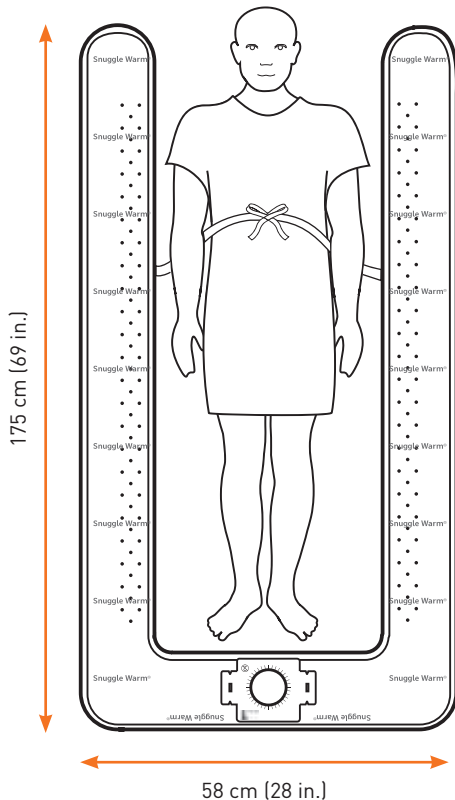

## Tube Blanket

58 x 175 cm


28 x 69 in.

### **Recommended for use in these procedures:**

- Interventional cardiology/cardiac electrophysiology
- Interventional radiology
- Trauma

### **Key attributes of the SWU-2007:**

- Allows for full access to the patient

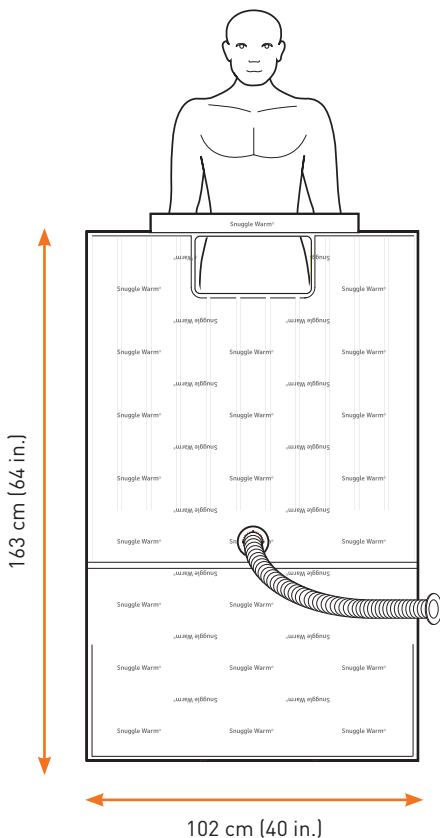

## Sterile Cardiac Blanket

102 x 163 cm

40 x 64 in.


Includes non-inflating foot drape

### **Recommended for use in these procedures:**

- Cardiac bypass

### **Key attributes of the SWU-2008:**

- Blanket can be used in the sterile field to warm the patient
- Groin window allows for visibility of femoral artery
- Sterile hose allows convective warmer to be positioned farther from the sterile field

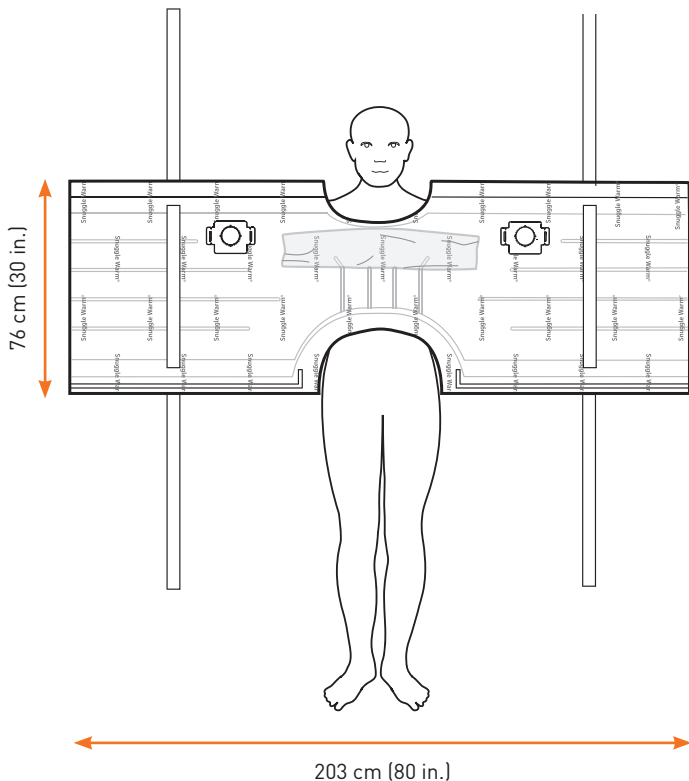

## Small Upper Body Blanket

203 x 76 cm

80 x 30 in.


### **Recommended for use in these procedures:**

- Lower extremities (hips, legs, knees, ankles)
- Gynecology
- Urology
- Open abdominal
- Laparoscopic

### **Key attributes of the SWU-2010:**

- Tie-downs aid in maintaining the blanket's position
- Smaller size allows for increased access to the patient
- Clear plastic drape captures and delivers warm air around the patient's head



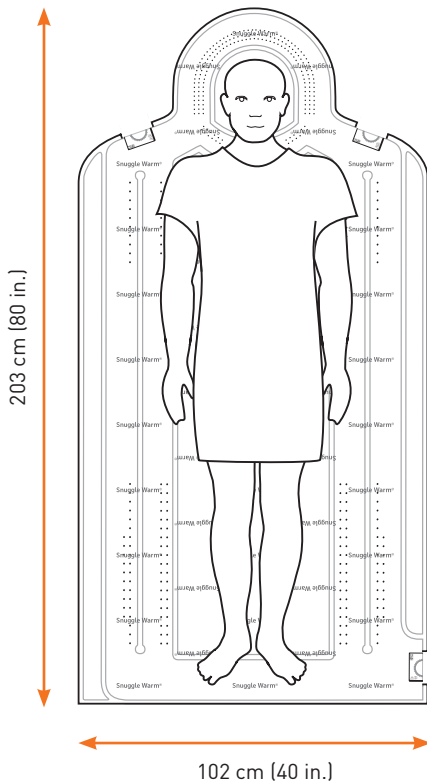

## Adult Underbody Blanket

102 x 203 cm


40 x 80 in.

### **Recommended for use in these procedures:**

- Cardiac
- Open abdominal
- Multi-site
- Cosmetic
- Trauma

### **Key attributes of the SWU-2013:**

- “Halo” design headrest area allows for warming of patient’s head
- Underbody blanket allows for full access to the patient
- Three hose ports for maximum convective warmer placement flexibility

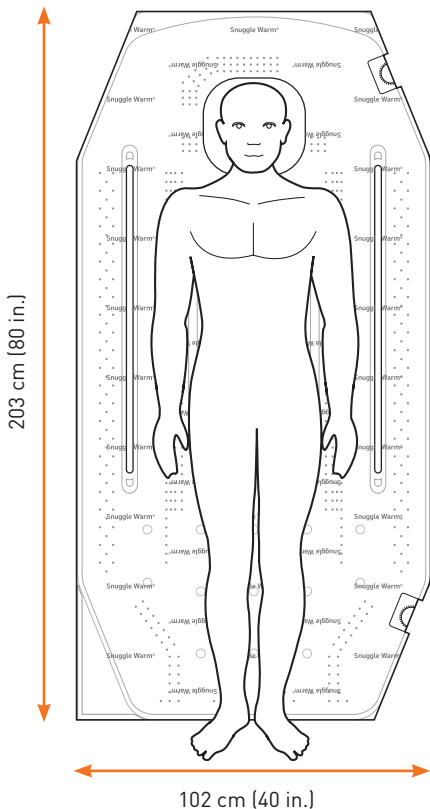

## Adult Underbody Blanket with Arm Openings

102 x 203 cm


40 x 80 in.

### Recommended for use in these procedures:

- Cardiac
- Orthopedic
- Cosmetic
- Open abdominal
- Multi-site
- Trauma

### Key attributes of the SWU-2113:

- Pre-opened arm and head openings save time
- Wide arm openings allow for easy positioning with arm(s) out or toboggan-style arm restraints
- Easily accommodates patients in a supine, lateral or prone position
- Tuck flaps secure the blanket to prevent sliding when transferring the patient to the operating table
- Underbody blanket allows for full access to the patient
- User-positioned head drape captures and delivers warm air around the patient's head and neck

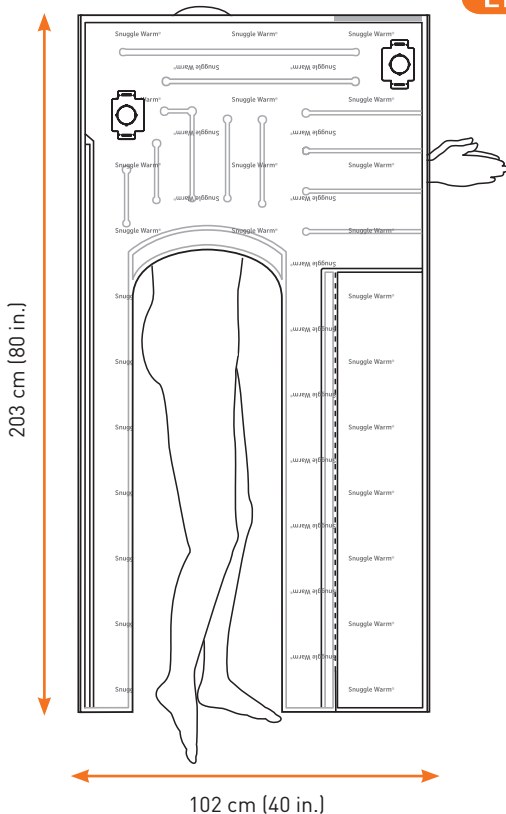

# SWU-2014L

## Left Lateral Access Blanket

102 x 203 cm

40 x 80 in.

### **Recommended for use in these procedures:**


- Hip
- Upper or lower leg

### **Key attributes of the SWU-2014L:**

- Proprietary design optimizes lower-body procedures



203 cm (80 in.)

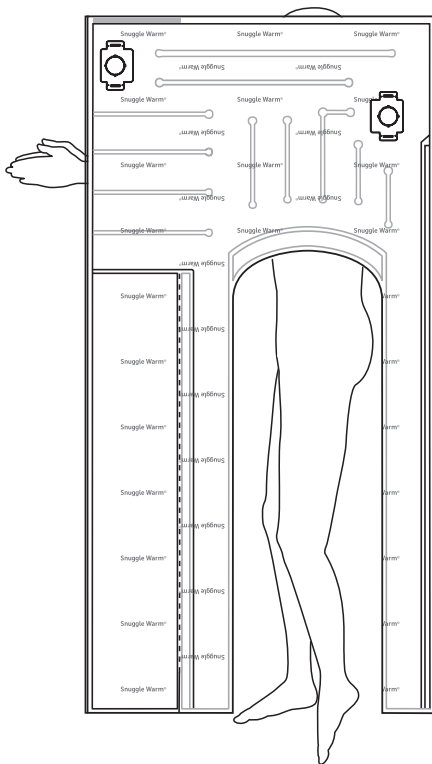

102 cm (40 in.)

## Right Lateral Access Blanket

102 x 203 cm

40 x 80 in.

### **Recommended for use in these procedures:**


- Hip
- Upper or lower leg

### **Key attributes of the SWU-2014R:**

- Proprietary design optimizes lower-body procedures



203 cm (80 in.)

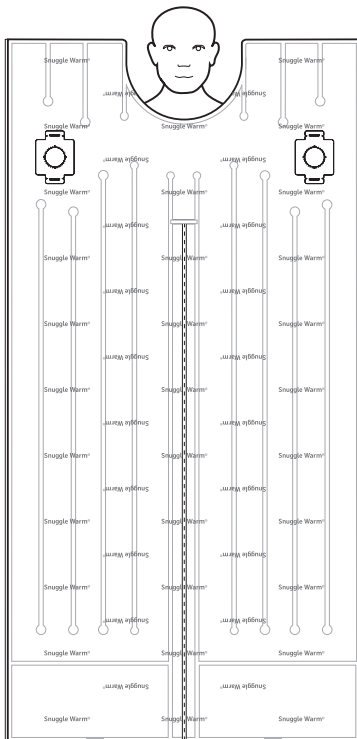

102 cm (40 in.)

## Full Body Split Access Blanket

102 x 203 cm


40 x 80 in.

### **Recommended for use in these procedures:**

- Knee, leg and foot
- Carotids
- Spinal
- Labor and delivery
- Gynecology
- Post-operative warming

### **Key attributes of the SWU-2016:**

- Perforation up the middle of the blanket allows for increased patient access

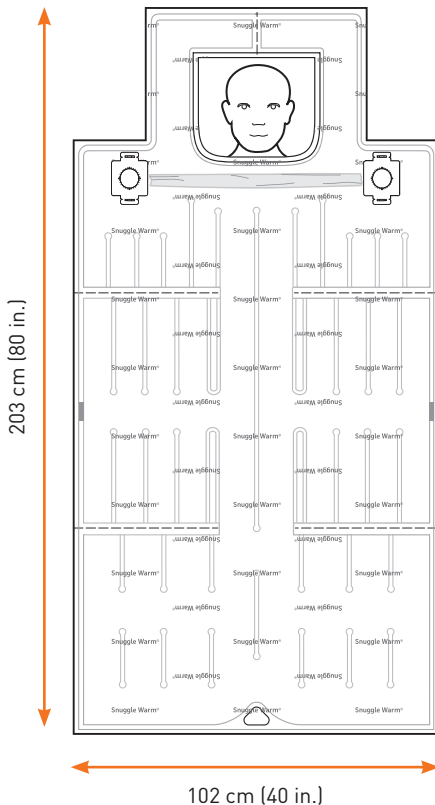

## Multi-Access Blanket

102 x 203 cm


40 x 80 in.

### **Recommended for use in these procedures:**

- Post-operative warming
- Catheterization lab
- “Keyhole” (minimally invasive) cardiac surgery
- Nephrectomy

### **Key attributes of the SWU-2018:**

- Perforations on either side of the blanket allow for increased patient access
- Clear plastic drape captures and delivers warm air around the patient’s head

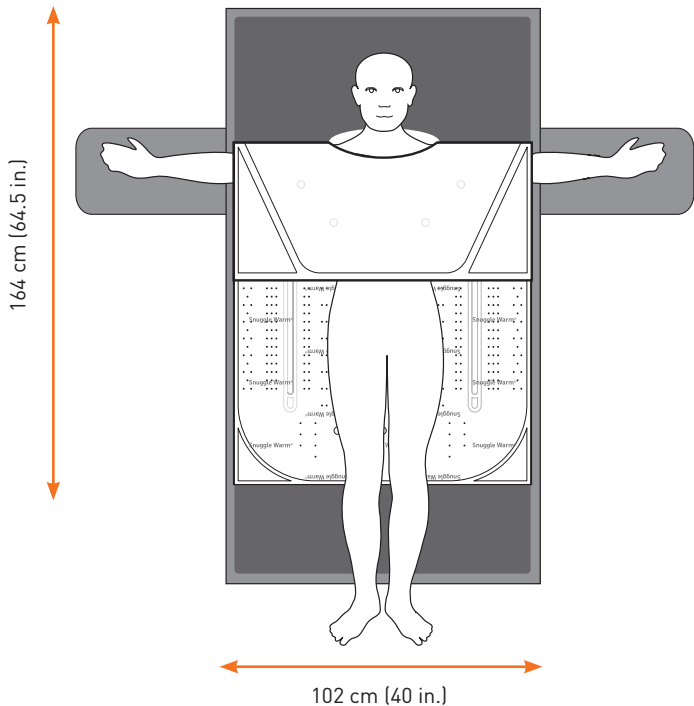

## Poncho Blanket with Arm Openings

102 x 164 cm


40 x 64.5 in.

### Recommended for use in these procedures:

- Urology
- Gynecology
- Labor and delivery
- Prostatectomy
- Radiology
- Abdominal
- Lower extremity (orthopedic, osteotomy, peripheral vascular disease, varicose vein, etc.)
- Pre- and post-operative warming

### Key attributes of the SWU-2119:

- Reversible design allows use pre-operatively and post-operatively as an overbody blanket and during surgery as an underbody blanket
- Underbody use during surgery enables simultaneous warming of the patient's backside and the front of the upper torso
- Wide arm openings permit easy positioning with arm(s) out or toboggan-style arm restraints
- Easily accommodates patients in the lithotomy or supine position

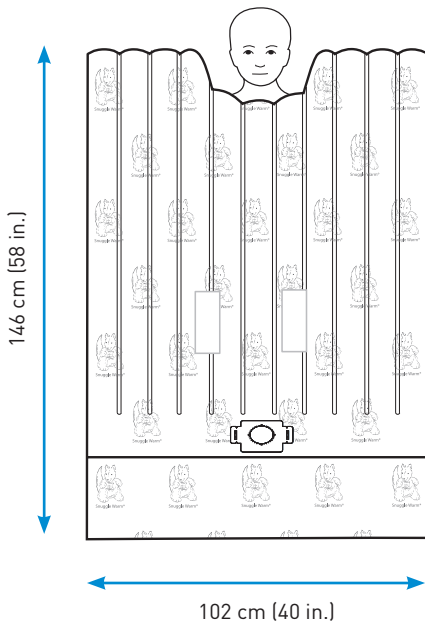

## Pediatric Full Body Blanket

102 x 146 cm

40 x 58 in.

Includes non-inflating foot drape


### **Recommended for use in these procedures:**

- Neurological
- Ear, nose and throat
- Pre-operative and post-operative warming
- Trauma

### **Key attributes of the SWU-2002:**

- Sized to fit pediatric patients



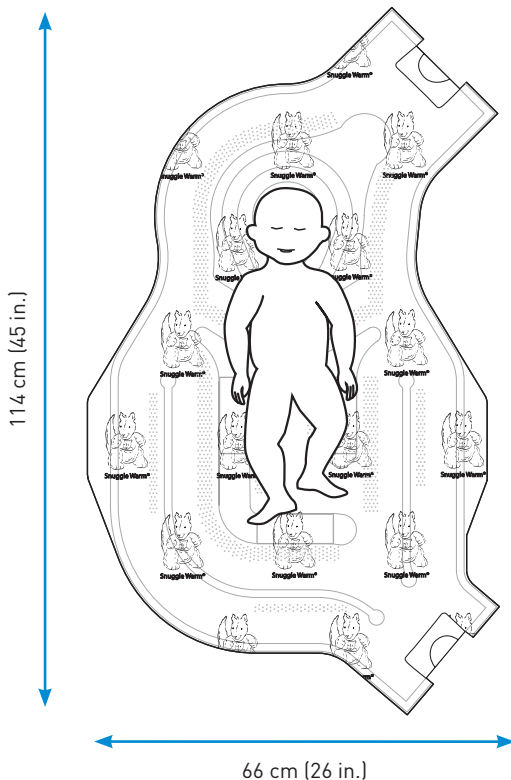

## Pediatric Underbody Blanket

66 x 114 cm

26 x 45 in.


Includes non-inflating flaps

### **Recommended for use in these procedures:**

- For patients up to 20 pounds (9.1 kilograms)

### **Key attributes of the SWU-2009:**

- Underbody blanket allows for full access to the patient
- Two user-positioned clear plastic drapes to capture warmth in specific locations while allowing access where necessary
- Uninflated headrest for better access
- Designed with absorbent pads to minimize pooling of fluid on the blanket
- Can be used on an operating table, bed, or bassinet

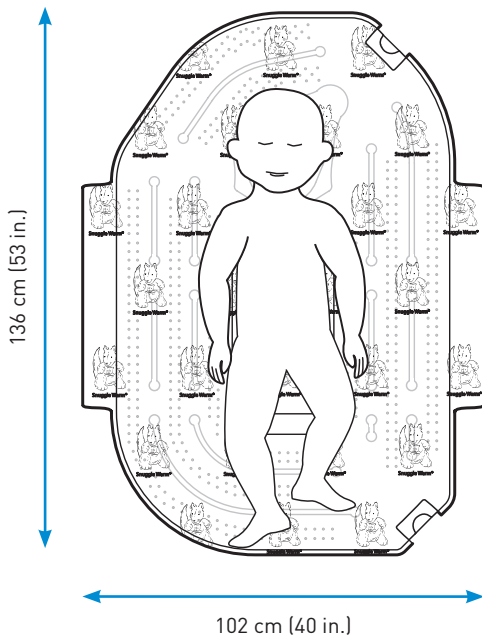

## Large Pediatric Underbody Blanket

102 x 136 cm

40 x 53 in.

Includes non-inflating flaps

### **Recommended for use in these procedures:**

- For patients up to 80 pounds (36.3 kilograms)

### **Key attributes of the SWU-2011:**

- Underbody blanket allows for full access to the patient
- Two user-positioned clear plastic drapes to capture warmth in specific locations while allowing access where necessary
- Designed with absorbent pads to minimize pooling of fluid on the blanket
- Can be used on an operating table, bed, or bassinet



**SWU-2001**  
Full Body  
Blanket



**SWU-2002**  
Pediatric Full  
Body Blanket



**SWU-2003**  
Upper Body  
Blanket



**SWU-2004**  
Lower Body  
Blanket



**SWU-2007**  
Tube  
Blanket



**SWU-2008**  
Sterile  
Cardiac  
Blanket



**SWU-2009**  
Pediatric  
Underbody  
Blanket



**SWU-2010**  
Small Upper  
Body Blanket



**SWU-2011**  
Large  
Pediatric  
Underbody  
Blanket



**SWU-2013**  
Adult  
Underbody  
Blanket



**SWU-2013**  
Adult  
Underbody  
Blanket with  
Arm Openings



**SWU-2014L**  
Left Lateral  
Access Blanket



**SWU-2014R**  
Right Lateral  
Access Blanket



**SWU-2016**  
Full Body Split  
Access Blanket



**SWU-2018**  
Multi-Access  
Blanket



**SWU-2119**  
Poncho  
Blanket with  
Arm Openings

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

**Smiths Medical ASD, Inc.**

St. Paul, MN 55112, USA

Phone: 1-614-210-7300

Toll-Free USA: 1-800-258-5361

[www.smiths-medical.com](http://www.smiths-medical.com)

Find your local contact information at:

[www.smiths-medical.com/customer-support](http://www.smiths-medical.com/customer-support)

Smiths Medical is part of the global technology business Smiths Group plc. Snuggle Warm, Level 1 and the Smiths Medical and Level 1 design marks are trademarks of Smiths Medical. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions.

©2015 Smiths Medical. All rights reserved. TM194827GL-052015

MPAUC-1182

**Rx**  
**ONLY**

**CE**  
**0473**

**smiths medical**